


Boston Scientific LUX-Dx™ myLUX™ patient app

Staying Connected

Keeping the myLUX patient app open and connected is crucial for your health care provider to access the latest data.




1

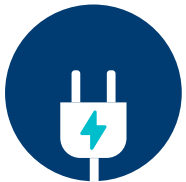

Always keep your smartphone charged, powered on, and connected to the internet via cellular signal or WiFi connection.

2

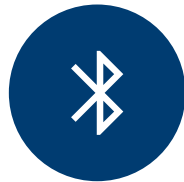

Keep Bluetooth® enabled on your phone, allowing the app to connect to your LUX-Dx ICM device.

3



Keep your smartphone within 2 metres of you while you are sleeping at night.

4



Unlock your phone and open the myLUX app. If you see a red icon, your app is not connected. Tap 'Check your connection' and follow the on-screen instructions. A green screen appears when your app is successfully reconnected. Tap 'Exit'.

5



Open your myLUX app every once in a while. Always look for a blue screen and a check mark icon after you open your app. **This confirms that your monitoring is up to date. No further action is needed from you.**

To ensure uninterrupted data transfer, remember these five key steps.



Best practices

Stay connected

- **Do not close the app** (i.e., do not swipe up)
 - If the app is closed for an extended period, you will get a notification – timing depends on your data transfer schedule.
- **Reopen the myLUX™ app** if you restart your smartphone.
- **Disable battery-saving features** that limit background activity (e.g., power saving, low power, battery optimisation, etc.)

Remain up to date

- **Keep notifications turned on** for your myLUX app to be updated in real time.
- **Keep background refresh and background data usage on.**



Need help?

To learn more about the myLUX app, scan this QR code to visit the myLUX tutorial website.



Contact Boston Scientific RhythmCARE™ Patient Services at RhythmCARE_EMEA@bsci.com, Monday to Friday 7:30 AM to 4:30 PM (GMT), or scan the QR code to get in touch with our RhythmCARE experts.

**Boston
Scientific**
Advancing science for life™

www.bostonscientific.eu

© 2025 Boston Scientific Corporation or its affiliates. All rights reserved.

CRM-2222602-AA

CE 2797

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at www.IFU-BSCI.com. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.